

*Reaching further patients in minutes  
at the point of care*

# HUBI DIA

Point-of-Care  
for Diabetes Management



✓ **Easy and Convenient**

Simple test procedure and result in **4 min.**

✓ **Accurate and Reliable**

IFCC and NGSP certified

✓ **Multiple assays on one platform**

HbA1c, ACR\*, CRP\* (*\*coming soon!*)



## Procedure



1 Open the lid on analyzer.



2 Insert the test cartridge.



3 Shake the reagent pack 3~4 times.



4 Collect the sample at sampling rod.



5 Insert the reagent pack in the test cartridge.




6 Close the lid. Result in 4 minutes.

## Specification

Principle	Boronate Affinity (BA)
Sample Type	Capillary whole blood Venous whole blood
Test Range	[NGSP] 4.0~14.0 % [IFCC] 20~130 mmol/mol
Storage Temperature	Test Cartridge: 2~32 °C (36~90 °F) Analyzer: 2~45 °C (36~113 °F)
Operating Temperature	10~40 °C (50~104 °F)
Size & Weight	(W)202 X (D)264 X (H)133 mm, 1.5 kg
Option	Touch LCD display, Internal printer, USB, TCP/IP - LIS/HIS compatible


## Performance

### Regression Analysis

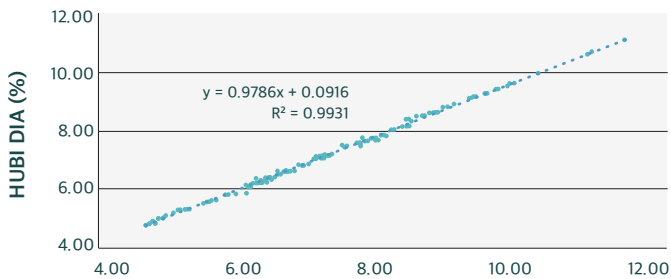

IFCC & NGSP (%)

### Accuracy Analysis

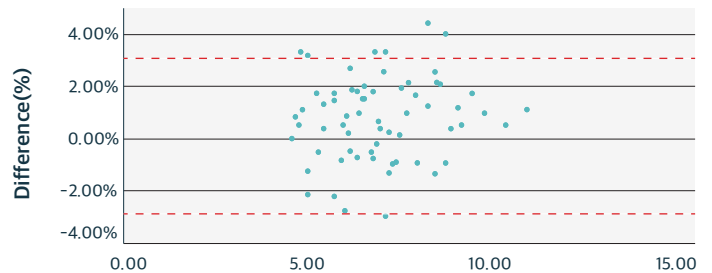

IFCC & NGSP (%)

## Ordering Information

Category	Cat. No.	Product Name	Package	Shelf Life
Analyzer	AHD-8001	HUBI DIA	1 Unit/Box	-
Test Kit	AHA1C-8010	HUBI A1c	10 Tests/Box	18 months



**Humasis Co., Ltd.**

114, 502, 504, 604, 604-1, B03-1, B03-2,  
88, Jeonpa-ro, Dongan-gu, Anyang-si,  
Gyeonggi-do, 14042, Republic of Korea  
TEL:+82-31-8085-6200, FAX:+82-31-8085-6249  
Email:question@humasis.com  
www.humasis.com



**Care for U Co., Ltd.**

K-tower Dong-guan 2F, iljik-ro 88, Manan-gu,  
Anyang city, Gyeonggi-do, Republic of Korea



**Meridius Medical Europe Ltd.**

Unit 3D, North Point House, North Point Business  
Park, New Mallow Road, Cork, T23 AT2P Ireland  
Tel: +353 (0)21 2066448.